

# Information

## Specifications

<b>CODE</b>	WA024
<b>HEIGHT</b>	81cm – 94cm (32” – 37”)
<b>WIDTH</b>	52cm (20.5”)
<b>DEPTH</b>	62cm (24”)
<b>SEAT HEIGHT</b>	55cm (22”)
<b>SEAT WIDTH</b>	40cm (16”)
<b>WHEEL SIZE</b>	11cm (4”)
<b>WEIGHT</b>	4.5kg
<b>MAX USER WEIGHT</b>	120kg (18.9st)

## Cleaning

The device must be cleaned regularly with a mild cleaning agent and a soft cloth.

NEVER use aggressive detergents.

Ask your retailer or the technical staff to check the walking aid regularly.

If the product is equipped with rubber ferrules, they must be checked and changed regularly.

## Materials

The product features an anodized frame and various plastic components

## Disposal

Dispose of this product in accordance with your local authority's waste disposal regulations.

## Warranty

This product has a twelve month warranty, from the date of purchase, against any manufacturing defect.

The Team at Drive DeVilbiss develops its products to give our customers the freedom to live independently. This encompasses their daily home life and provides them with the opportunity to enjoy an outing with family and friends. Our goal is to develop a range that will provide individuals with a chance to enjoy everyday life.

Issued (07/17)

Dealer Stamp:



Drive DeVilbiss Healthcare Ltd, Heathfield Lane,  
Birkenshaw, West Yorkshire, BD11 2HW

[www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk)



Lightweight Aluminium  
Walker with Wheels



User Manual

## Safety Indication



Please read these safety instructions carefully before use, and keep them for future questions.

- » ALWAYS consult with your occupational therapist or mobility supplier to determine correct adjustment and use of the device.
- » DO NOT store walker outside or leave in the sun.
- » DO NOT leave the product in a car in case of high outside temperatures.
- » DO NOT use in extreme temperatures (above 38°C or 100°F or below 0°C or 32°F).
- » Please inspect all parts (screws, tubes, wheels, handles and frame) prior to use.
- » Check regularly if all screws are tightened.
- » Hold handles firmly when using the walker
- » In case of falls: let the product fall sideways to avoid falling on it.
- » Always move forward gently and with short steps to ensure a good balance of your weight and to avoid instability.
- » DO NOT use on wet or icy surfaces (snow, ice, water).
- » DO NOT use on stairs.
- » Use ONLY as a walking aid.
- » DO NOT exceed the weight limit of the walker.
- » After unfolding the walking aid, ensure that it is securely locked in the OPEN position and on a level surface.
- » Always test to see that the walking aid and attachments are securely locked in place before use.
- » DO NOT hang anything on the walking aid.
- » DO NOT modify the device; doing so will invalidate the warranty.
- » Original parts must be used to replace broken or defective parts.
- » If the product has not been used for a long period of time, check thoroughly for rust or corrosion prior to use.
- » Be careful: When using the walking aid, beware of holes and deformations in the floor.
- » Be careful: Risk of crushing or squeezing fingers when folding / unfolding the walker.

## Installation

### Set up

Only a competent person can adjust the product and teach you how to use it. Wrong usage or adjustment can lead to overstress of the device and injury to the user.

Ask technical staff how to use this product and make sure that you clearly understand their explanation. In case of doubt, do not hesitate to ask again.



### Positioning the handles

Position the anatomic push handle so that when the user's arm is down by their side, the handgrip is at wrist height.

Install the adjustment knob into one of the adjustment holes and tighten by turning clockwise (5).

### Opening

Pull the legs of the rollator backwards (1). Check that the locking mechanisms located on each side of the walker are horizontal and correctly locked (2). Check stability before using.

### Folding

To fold the rollator, lift up the seat (3) and pull up on the strap under the seat until the rollator closes (4).

