

drive

DeVilbiss
HEALTHCARE



All Terrain Cane
Instruction Manual

SAFETY SUMMARY

To ensure the safe operation of the Drive Canes and minimise the risk of injury, these instructions and safety precautions **MUST** be followed. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury may occur.

GENERAL WARNINGS

The All Terrain Cane is a walking aid intended to assist those who have reduced mobility. This cane can provide ambulatory assistance for an individual weighing up to 113 kg (18 st). Over-loading the cane is the only known contraindication, although others may be specific to an individual or care environment.

Safety Precautions:

- Always consult a medical professional to determine proper adjustments and usage.
- Before using the cane, ensure that it is securely locked in place or a fall could result, causing bodily injury and / or damage to the cane.
- Not intended to support the total weight of the user.
- Not intended for use on ice or other slippery surfaces.

FEATURES

- Aluminium cane material
- Retractable stainless steel spikes for slippery surfaces
- Height adjustable between 76.5-99 cm (30-39")
- Foam hand grip
- Wrist strap
- Easy to wipe clean

PROPER CANE HEIGHT

For a more comfortable fit, ensure your shoe height is the same as the style of shoes worn most frequently when using the cane.



1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
- 2 Adjust the cane height so the top portion of the handle touches inside of your wrist, where it meets the palm.


The retractable spike section may need to be removed to fully adjust the cane to its lowest height.

HEIGHT ADJUSTMENT

1. Turn the anti-rattle collar **ANTICLOCKWISE** to loosen.
2. Press **IN** the snap button.
3. Slide the extension up or down until the snap button protrudes through an adjustment hole on the handle end of the cane.

NOTE: An audible “CLICK” will be heard when the snap button securely protrudes through the adjustment hole.

4. Repeat **STEPS 2-3** until the cane is at height determined by **PROPER CANE HEIGHT** in this instruction manual.
5. Tighten the anti-rattle collar by turning **CLOCKWISE**.

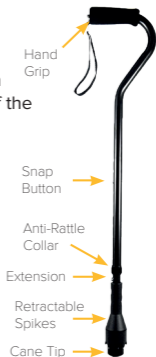

PRODUCT CARE

- Inspect cane tips for wear and damage. Replace any broken, damaged or worn items or parts immediately. Part codes for the replacement tips are included in this manual.
- The retractable spikes may be removed for storage or to fully adjust the cane height. No other parts of the cane should be folded or dismantled during use, transport or storage.
- Ensure that the cane is securely locked in position, with the snap button fully protruding through the adjustment hole. The snap button should release and re-engage properly.
- The cane can be cleaned with a damp cloth and mild soap solution. Avoid neat bleach or abrasive surface cleaners. Leave to fully dry before reuse.

SURFACES AND RETRACTABLE SPIKES

The All Terrain Cane is intended for frequent indoor or outdoor use, and is designed for use on most surfaces and under most weather conditions. However, we recommend that particular care is taken on any unfamiliar surfaces.

To extend or retract the spikes, push the snap button near the tip and lock in position, similar to the HEIGHT ADJUSTMENT. Always check the tip of the cane is secure, ensuring the spikes are not loose.

To prevent damage to flooring, only use the steel spikes outdoors and retract when not in use.



REPLACEMENT TIPS

Part code:

RTL10322BKB

Black Rubber Tip

1pc

Spare parts are available for maintaining your cane. To order replacement tips or any other parts, contact your provider, using the codes provided.

Visit www.drivedevilbiss.co.uk to find your local stockist. Spare part catalogues and other documentation can also be found on this website. Only use parts approved by Drive DeVilbiss Healthcare Ltd.

SPECIFICATION

Model	10306AT-6
Weight capacity	113 kg (18 st)
Height with All-Terrain Tip extended	76.5–99 cm (30-39")
Height with All-Terrain Tip retracted (rubber tip exposed)	80–98 cm (31.5-38.5")
Height With All-Terrain Tip removed	75–98 cm (29.5-38.5")
Colour	Black
Material	Aluminium
Handle type	Foam Grip

MAINTENANCE

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product.

Replace the tip every six months, or when damaged.

When the product has reached the end of its useful life, follow local recycling and disposal policies.

If you experience problems, please contact your provider or visit drivedevilbiss.co.uk for more info.

WARRANTY

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorised Drive dealer.

For more information, or assistance with use and maintenance, please contact your provider. Users with visual, reading or cognitive disabilities should seek advice from their provider or a professional care provider for an appropriate format. For the latest version of this document, contact Drive DeVilbiss Healthcare Ltd using the details below, or check our website.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of your Member State. Please quote the product serial code on all correspondence, which can be found on the cane label.

**Com-Da Healthcare Co., Ltd**

No. 1, Jinzhao Street, West District, Zhongshan City,
Guangdong Province, CN

**Osmunda Medical Technology Service GmbH**

Treskowallee 108, 10318 Berlin, DE

**Drive DeVilbiss Healthcare Ltd**

Sidhil Business Park, Holmfild, Halifax,
West Yorkshire, HX2 9TN, Great Britain

Tel: +44 (0) 845 0600 333 | info@drivedevilbiss.co.uk

Drive DeVilbiss Sidhil Ltd.

4 Trench Road, Mallusk, Newtownabbey,
BT36 4TY, Northern Ireland

Drive DeVilbiss Healthcare Ltd provide these instructions for use and product markings.

