



The folding commode has a robust yet lightweight steel frame with a 120kg (20 stone) weight capacity. It features an easy to use folding mechanism and plastic seat, armrest and backrest. It comes with a 5 litre potty and lid.

TO ASSEMBLE THE COMMODE

Holding the back of the commode, pull down on the front of the seat. The front and rear of the frames will separate. Once folded out, secure the frame by locking the plastic catches and both sides of the commode (see photograph below):



Slide the potty (and lid if required) underneath the seat from behind (See photo), ensuring that the potty is securely placed on the two rails underneath the seat.



TO FOLD THE COMMODE

Remove the potty by pulling it out from underneath the seat. Unlock both catches and pull up on the front of the seat whilst holding the back.

OPERATION

- The commode has two sturdy arm rests to aid users when sitting or getting up from the commode.
- Please see previous sections for details on potty removal and insertion.
- The potty comes with a lid to be used when transporting the potty.

CLEANING INSTRUCTIONS

The commode should be wiped clean with hot water and a mild detergent or disinfectant. The bucket and lid can be removed for cleaning. The plastic parts of the commode may also be cleaned with a hosepipe or similar equipment if required.

SAFETY CONSIDERATIONS

- The maximum weight capacity of the commode is 120kg (19 stone).
- For the best stability try to ensure the users weight is distributed as centrally and evenly as possible across the commode.

SPECIFICATION

Normal dimensions: 49(D) x 54(W) x 77(H)cm

Folded dimensions (exc potty): 15(D) x 54(W) x 78(H)cm

Seat dimensions: 38(D) x 40(W) x 45(H)cm

Weight: 5kg Potty capacity: 5 litres

Weight capacity: 120kg (19 stone)

WARRANTY

The stacking commode comes with a twelve month warranty from date of purchase. The warranty covers design and manufacturing defects. It does not cover environmental damage, accidental damage or wear and tear. For full details of the warranty please contact your dealer. The commode is manufactured by:

Drive DeVilbiss Healthcare Ltd
Heathfield Lane, Birkenshaw
West Yorkshire, BD11 2HW
www.drivedevilbiss.co.uk