

## Specification

Maximum user weight - 135kg  
Seat height range - 34cm-52cm  
Seat width - 30cm  
Product weight - 2kg  
Overall width - 33cm

## Maintenance

To maintain the quality of finish, we recommend that the bath stool is wiped down and dried after each use. Periodically, clean the stool with a damp cloth and mild detergent or disinfectant. For example, chloride disinfectant (solution of 1000 parts per million) can be used. Check for any damage to the seat and/or legs, if in doubt, remove from service.

## Warranty Details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement. If you have any queries or require full warranty conditions, please contact your Drive stockist. It is recommended that this stool is serviced on an annual basis, if the stool is used by a single user. If the stool is used in a multi-user environment (such as in communal housing) servicing may be required more frequently.

Any servicing, maintenance or repairs should be done by competent personnel. If you have any queries or require full warranty conditions, please contact your Drive stockist.



**Drive DeVilbiss Healthcare Ltd.**  
Halifax, W. Yorkshire, Great Britain

Tel: 0845 0600 333 | Fax: 0845 0600 334  
Email: [info@drivedevilbiss.co.uk](mailto:info@drivedevilbiss.co.uk)

[www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk)

The Drive logo is a stylized, lowercase blue font with a yellow accent on the 'i'.

The DeVilbiss Healthcare logo features a blue square icon with a white stylized 'D' shape inside, followed by the text 'DeVilbiss' in a bold blue font and 'HEALTHCARE' in a smaller, all-caps blue font below it.




# BATH STOOL WITH SWIVEL SEAT

User Manual

# Safety Precautions

1. This device features a freely rotating seat.
2. This device is only suitable as a seat. Do not stand.
3. Make certain that the height adjusting push buttons fully protrude through the same respective hole of each leg extension. This will ensure that the legs are securely locked in position and an even height is achieved.
4. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace if any of these imperfections exist.
5. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.

# Assembly

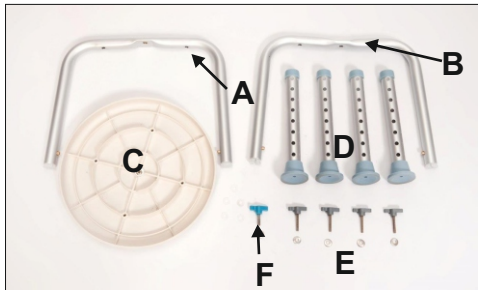

- A: Leg Section A (x 1)
- B: Leg Section B (x 1)
- C: Rotating Seat (x 1)
- D: Height Adjustable Legs (x 4)
- E: Grey Handscrew with metal washer (x 4)
- F: Blue Handscrew with metal washer (x 1)

## Step 1.

Place the seat upside down, and secure the leg section (A) using 2 grey handscrews with a metal washer.



## Step 2.

Add leg section B over section A, and secure the leg section (B) using 2 grey handscrews with a metal washer



## Step 3.

Attach the blue handwheel and metal washer (F) into the centre of the frame to secure the frame.



## Step 4.

If required insert the legs (D) on to the frame whilst pressing the brass push button on the frame. Press the brass push button to allow the leg to move then position until it locks at the correct height. Repeat for the other 3 legs, ensuring that all legs are set to the same height.

