

Aluminium Adjustable Swivel Bath Seat

The Adjustable Swivel Bathing Seat is constructed from corrosion free aluminium and features a swivelling plastic seat and removable armrests.

It is designed for a single user up to 160kg (25 stone), and aids the user to compensate for a lack of mobility.


The swivel bath seat will fit most standard UK baths. The dimensional parameters for the bath are in the specification section.

Safety Precautions

Apply all of the following precautions before use:


- Inspect all parts for shipping damage before assembly. If there is shipping damage – DO NOT USE. Contact your dealer for further instructions.
- Ensure that all four 'E' clips are fully installed on the traverse arms.
- Check all four corners of the swivel bath seat are resting on the top edge of the bath by at least 5cm (2"). Check that the setscrews are lightly touching the inside of the bath. Do not over tighten the setscrews.
- Always sit in the centre of the swivel bath seat, and do not lean on the edges of the bath seat as this can cause instability. DO NOT stand on the bath seat.
- Do not exceed the weight limit of 160kg (25 stone).

Contents

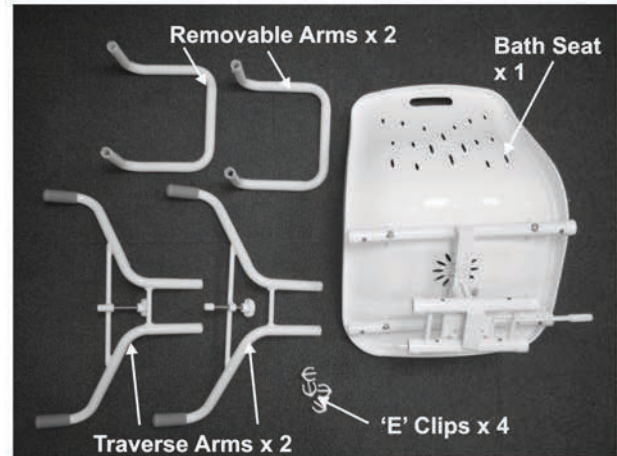

Operation

Refer to the Assembly Instructions Supplied Before Operating the Swivel Bath Seat

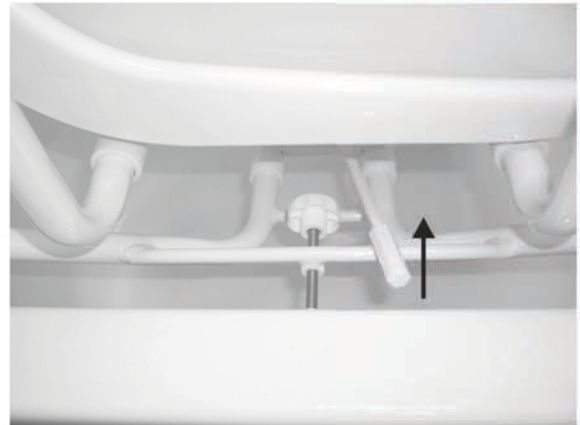

1. Lift up the lever located underneath the right hand side of the seat, and swivel the seat. Release the lever and keep swivelling the seat until it locks in position.



2. Position the seat so it is facing the user (as shown below)



3. The user should safely sit on the seat in the position above. The swivel lever can then be operated to allow the seat to be positioned in the bath (as shown below). The user can now bathe.



4. After the user has bathed, the chair can be swivelled to the facing position for safe exit from the bath.

Care and Maintenance

- To maintain the quality of finish, we recommend that the swivel bath seat is wiped down or showered and dried after each use.
- Periodically, clean the bath seat with a damp cloth and mild detergent or disinfectant. For example, a mild chloride disinfectant (solution of 1000 parts per million) can be used.
- The bath seat can also be cleaned in an aqueous system (for example a Turbex machine) up to 80°C.
- Check that the bath seat swivels easily, smoothly and locks in to place. If the locking mechanism does not feel positive or there is excessive movement in the device, remove from use and contact your dealer.
- Check for cracks and damage to the bather, in particular the plastic seat and tubing.
- Do not use in environments over 60°C (saunas, for example).

Specification

Fits standard baths with:

Outside width 69 – 74cm (27" – 29"), &

Inside width 50 – 61cm (20" – 24")

Overall Dimensions:

55cm (D) x 69 – 74cm (W) x 42cm (H)

22" (D) x 27" – 29" x 17" (H)

Seat Dimensions:

42cm (D) x 46cm (W) x 5cm* (H)

17" (D) x 18" (W) x 5"* (H)

Armrest Height 23cm (9")*

Weight 2kg

Weight Limit 160kg (25 stone)

'' refers to the height from the top of the bath

Warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

Manufacturer Details



Drive DeVilbiss Healthcare Ltd,
Heathfield Lane, Birkenshaw,
West Yorkshire, BD11 2HW


Tel: 0845 0600 333 | Fax: 0845 0600 334

Email: info@drivedevilbiss.co.uk


www.drivedevilbiss.co.uk

Appendix : Assembly Instructions


1. Press the two brass push buttons whilst inserting the arm rest in to the bath seat frame. Release the buttons and carry on insert the arm rest until it locks in to position with an audible 'click'

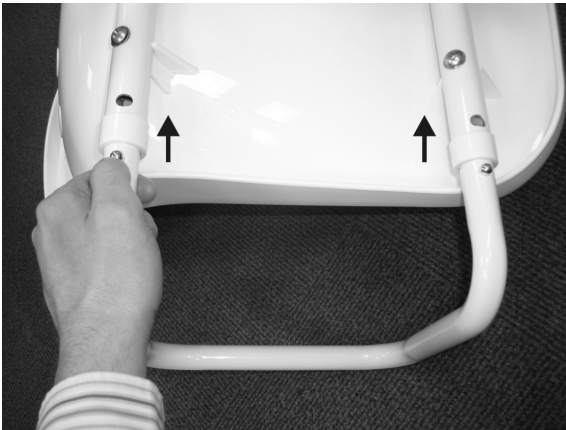

2. Insert the two traverse arms in to the frame and line up the holes when the desired width is reached.

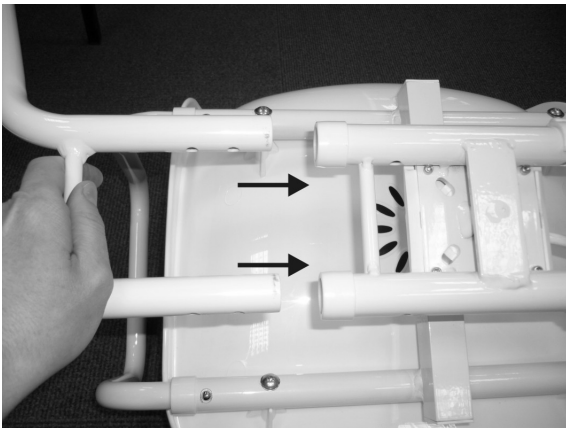

3. Secure the traverse arms using the four 'E' clips supplied. Ensure the 'E' Clips are fully inserted and the ends protruding through the tubing.

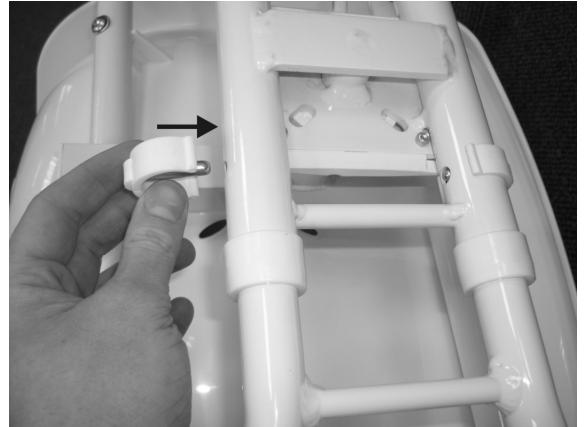

4. Place the bath seat in the bath as shown below.



5. Turn the setscrews as shown below until they lightly touch the side of the bath. Do not over tighten the setscrews as this could damage the bath.

