care and maintenance

- To maintain the quality of the finish, we recommend that the bench is showered or wiped down and dried after each use.
- Periodically clean the bath bench with a damp cloth and mild detergent or disinfectant. Do not use abrasive detergents as these can damage the plastic parts.
- Check that all screws, nuts and /or bolts are tightened.
- Check the rubber ferrules on the legs for rips, tears or wear. Replace if the ferrules are damaged, worn or missing.

specification

The bath bench with back is constructed from aluminium tubing, PE-HD white plastic seat and back, and rubber ferrules.

Overall dimensions (chair at its highest setting):
53cm (D) x 50cm (W) x 82cm (H) /
21”(D) x 20”(W) x 32”(H)

Footprint (chair at its highest setting):
52cm (D) x 46cm (W) / 21”(D) x 18” (W)

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Weight Capacity: 21 stone (135kg)

warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB HX5 9JP
www.drivemedical.co.uk

Document code: 12202KD_Hangtag

bath bench with back

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.co.uk



safety precautions

Apply the following precautions before use:

- Inspect all part for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact your dealer for further instructions.
- Make certain that all the brass push buttons snap to lock in position and protrude through the holes on the legs.
- Check that all screws, nuts and /or bolts are tightened.
- Check the rubber ferrules on the legs for rips, tears or wear. Replace if the ferrules are damaged, worn or missing.
- All four legs must touch the floor simultaneously before use.
- DO NOT stand on the bench.
- Always sit in the centre of the bench, and do not lean on the edges of the bench as this can cause instability.
- Do not exceed the maximum weight limit of 135kg (21 stone).
- The bench is only suitable for use by one user at a time.


assembly instructions

1. Remove contents from box.
2. Lay seat upside down on a flat surface so the 4 leg receptacles are facing upward.
3. Insert first leg into the leg receptacle by depressing push pin. When the leg is in place, the brass push pin will extend out and lock the leg in place (Figure 1).
4. Repeat Steep 3 for the three remaining legs, ensuring all legs are set at the same height.
5. Attach backrest to seat by inserting both back support tubes into the back support receptacle on the seat. The brass push buttons will snap out and lock the back support in place (Figure 2).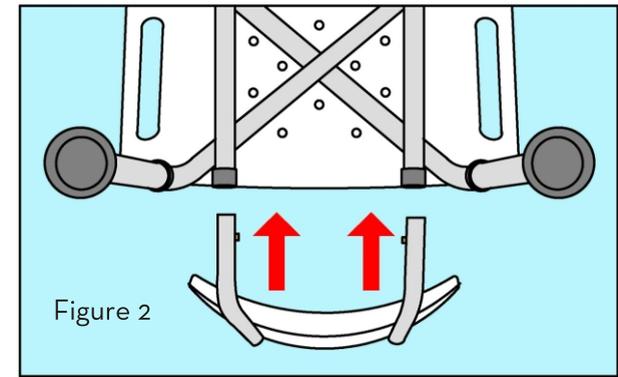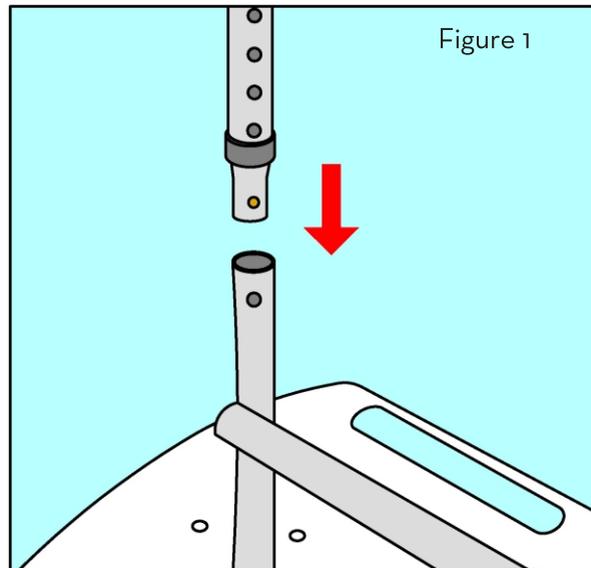

height adjustment

To adjust the seat height, use the push button on leg section of the chair. Ensure that all legs are set at the same height.

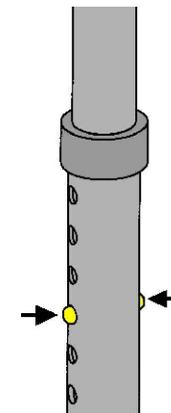To adjust the leg height:

1. Push in the brass buttons (as shown in the diagram).
2. Extend or retract the section to the required height.
3. When this height is reached, release the button but carry on adjusting the section until the buttons lock in position.
4. Repeat for the other 3 legs.

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