drive  
WWW.DRIVEMEDICAL.COM

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EU Authorized Representative  
Drive Medical Ltd,  
Ainley's Industrial Estate,  
ELLAND HX5 9JP, GB

drive  
WWW.DRIVEMEDICAL.COM

## assembly instructions

1. Remove contents from box.
2. Turn seat section upside down. Unfold legs and secure leg section 2 (marked on product) into metal receptacle on bottom of the chair. Turn seat right side up and press down on seat until leg section 2 is fully secured.
3. Insert leg extensions into the corresponding seat sections by matching up the 1's and 2's on the seat section with the 1's and 2's on the legs.
4. Leg height is easily adjusted by depressing brass push pins and setting to desired height.

Periodically check assembly for continued tightness. Never use bath safety seat if any looseness in the assembly is noticed, as personal injury may result. Sit, do not stand on this seat to avoid a possible dangerous fall.

**NOTE:** To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

**WEIGHT CAPACITY: 300 LBS.**

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

**drive**  
WWW.DRIVEMEDICAL.COM

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.