

PRODUCT INSTRUCTION SHEET

REFERENCE: Oakham Chair ~ '700' (with winged back) '700NW' (without winged back)

DESCRIPTION

- Brown vinyl, metal frame, with or without winged head support and with lumber support.
- Generously padded and covered in vinyl for easy cleaning.
- Completely adjustable (overall height, seat depth and base height).



Overall height: 42.5" – 47.5" (1080mm – 1206mm)

Width between arms (outside): 22.5" (572mm)

Width of Back with Wings: 24"(635mm), w/o Wings: 19.5" (495mm)

Overall depth: 24" (610mm)

Seat height min-max: 16.5" – 21.5" (420mm – 550mm)

Seat depth: 17.5'' - 20.5''Seat Width: 19.5'' (495mm)

Back height: 26'' - 28.5'' (660mm - 724mm)Arm height: 25'' - 30'' (635mm - 762mm)

USAGE REQUIREMENTS

- 1. When adjusting the legs, ensure that there is no gap between horseshoe clip and frame.
 - The clip spindle must be fully located right through the outside frame.
- 2. Check that the two wing nuts are fully tightened on the seat base and back support.
- 3. Do not exceed user weight of 165kg (26 stone).
- 4. Clean the frame with cream cleaner and upholstery with proprietary polish as required due to usage.



