

PRODUCT INFORMATION SHEET

REFERENCE

Stacking Commodes

180 (Fixed Height)

181 (Adjustable Height)

182 (Adjustable Height with Detachable Arms)

DESCRIPTION

All welded coated tubular frame with cushioned over seat and back rest covered in flame retardant, waterproof vinyl.

Plastic hinged toilet seat and 3.5 litre pan and lid.



TECHNICAL SPECIFICATIONS

Overall dimensions (181/182)	55cm (W) x 55cm (D) x 77-92cm (H)
Seat dimensions (181/182)	36cm (W) x 36cm (D) x 45-60cm (H)
Armrest Height (181/182)	64 – 79cm (H)
Overall dimensions (180)	55cm (W) x 55cm x 77cm (H)
Seat dimensions (180)	36cm (W) x 36cm (D) x 46cm (H)
Armrest Height (180)	65cm (H)
Weight Limit	180kg (28 stone)
Weight	6.5kg



USAGE REQUIRMENTS

1. Do not exceed recommended weight of 180kg (28 Stone)
2. The product should not be “swung on” in a forward or backward motion which could render the product unstable and could cause the legs to bend.
3. Where legs are to be adjusted, ensure that all legs are at the same height and the E-clips are pushed right through to the outer frame.
4. Periodically check that the vinyl covered over seat and backrest do not show any signs of damage e.g. tears, as they could become contaminated.
5. The commodes are designed for minimum maintenance. They require the occasional wipe over with a soft cloth using neutral detergent and warm water. Disinfect with chloride disinfectant – rinse and dry.
6. Periodically check for elongated or stressed adjustable holes. Check for split, worn or loose ferrules. Check that all fittings are secured, and there is no damage to the outer coating of the frame.

