


## Height Adjustment

The height of the transfer bench can be adjusted to suit the user and the bath tub being used.

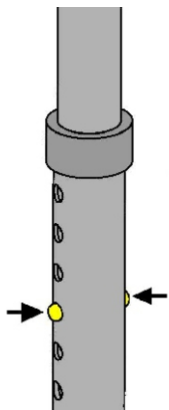

To adjust the leg height:

1. Push in the buttons (as shown in the diagram).
2. Extend or retract the section to the required height.
3. When the height is reached, release the buttons but carry on adjusting the section until the buttons lock into position.
4. Repeat for the other three legs.

## Using the Transfer Bench

**NOTE:** The procedures below are guidelines. Please follow the recommendations from your healthcare professional regarding transferring.

1. Position the transfer bench with two legs in the bath and two legs outside the bath, with the handle inside the bath.
2. The user should sit on the bench, without getting into the bath.
3. To transfer into the bath, the user should lift their legs and place them in the bath, whilst moving across the seat, until they are sat over the centre of the bath.
4. Reverse the above procedures when exiting the bath.

## Care and Maintenance

To maintain the quality of the finish, we recommend that the bench is showered or wiped down and then dried, after each use.

Periodically clean the transfer bench with a damp cloth and mild detergent or disinfectant. Do not use abrasive detergents, as these can damage the plastic parts.

Regularly check that all screws, nuts and bolts are tight and secure.

Check the rubber tips and suction cups on the legs for tips, tears or wear. Replace if the parts are damaged, worn or missing.

## Specification

Maximum User Mass: 135kg (21 stone)

Seat Width: 66cm  
Seat Depth: 50cm  
Seat Height: 46 - 61cm

Overall Depth: 61cm  
Overall Width: 84cm

## Warranty Details

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

This does not affect your statutory rights.



Drive DeVilbiss Healthcare Ltd.  
Sidhil Business Park, Holmfield, Halifax,  
West Yorkshire, HX2 9TN, Great Britain

Drive DeVilbiss Sidhil Ltd.  
4 Trench Road, Mallusk, Newtownabbey,  
BT36 4TY, Northern Ireland

[www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk)

Issue: IFU - RTL12011KDR Transfer Bench (2022-03 Rev2)



## Transfer Bench

### User Manual

## Safety Precautions

Fully read these instructions before assembling or using the transfer bench.

Check that you have all the parts and that they are free from damage, before starting assembly.

Ensure that all push buttons snap into position and protrude through the holes in the legs.

Check that all fixings are tight and secure.

Check the rubber tips and suction cups for rips, tears or wear. Replace them if the parts are damaged, worn or missing.

Ensure that all four legs touch the floor/bath, before use.

Do not stand on the transfer bench.

Always sit on the seat of the bench. Do not lean over the edges of the bench, as this can cause instability.

Do not exceed the Maximum Weight Capacity of 135kg (21 stone).


The bench is only suitable for one occupant at a time.

## Assembly Instructions

1. Remove all contents from the carton.
2. Lay the bench upside down, on a flat surface.
3. Attach leg with large suction cup (1) by depressing the push buttons and inserting into the receptacle (1). Repeat for the second leg.




4. Attach leg with small rubber tip (2) by depressing the push buttons and inserting into the receptacle (2). Repeat for the second leg.

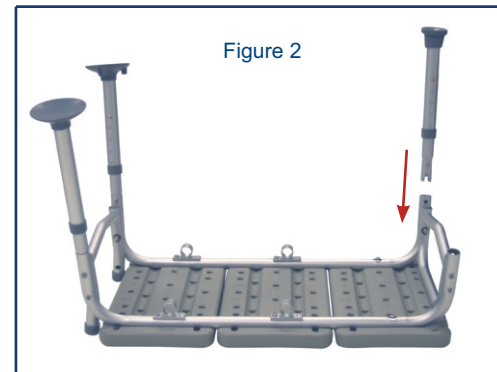

5. Turn the transfer bench the right way up. Attach the armrest by depressing the push buttons and sliding it into the arm receptacle, on the end of the bench. Ensure the armrest arcs outwards, not inwards.




6. To assemble the backrest, insert the pole into the backrest support by depressing the push buttons and sliding it into the receptacle. Repeat for the second pole.

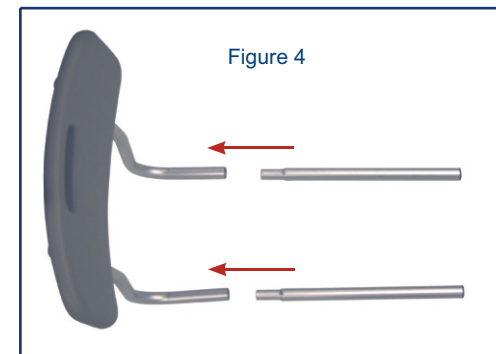

7. Insert the backrest into the base by depressing the push buttons and sliding the supports through both receptacles.

Note: Backrest must go through both sets of receptacles to become secure.

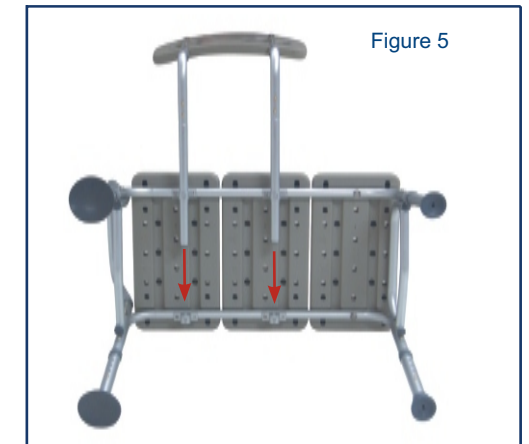

Note: The backrest is reversible to accommodate left or right tub entry. You can change the seat depth of your transfer bench by pushing the buttons under the seat and sliding the poles forwards, until the buttons lock into place.

8. Ensure the legs with suction cups are placed into the bath tub.

