

# Warranty Details

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

This does not affect your statutory rights.



Drive DeVilbiss Healthcare Ltd.  
Sidhil Business Park, Holmfield, Halifax,  
West Yorkshire, HX2 9TN, Great Britain

Drive DeVilbiss Sidhil Ltd.  
4 Trench Road, Mallusk, Newtownabbey,  
BT36 4TY, Northern Ireland

[www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk)

Issue: IFU - SWALL002 Shower Stool (2022-03 Rev2)

The word "drive" in a blue, lowercase, italicized font.

The "DeVilbiss" logo (a yellow square with a blue 'D' shape) and the word "HEALTHCARE" in a blue, uppercase, sans-serif font.



## Shower Stool Wall Mounted

### User Manual

## Introduction

The Drive DeVilbiss SWALL002 Folding Shower Stool is constructed from aluminium and features height adjustable drop-down legs, for support.

It features a HD polyethylene seat, with drainage holes and folds away to the wall to save space, when not required.

Maximum Weight Capacity: 160kg (25 stone)

## Installation

The Shower Stool must be permanently fixed to a wall, before use. It must be installed by a competent person and only fastened to a wall which has the capacity to withstand the vertical load and leverage applied during use. The mounting plate features six 8mm diameter holes. Fasteners are not supplied because the method of fixing will depend on material and thickness of the wall. The person fitting the seat will be able to supply suitable fasteners.

Prior to fixing the plate to the wall, the leg height needs to be set to suit the user. To set the leg height, pull out the E-Clip, slide the leg to the required position and then re-insert the E-Clip. Repeat for the other leg, ensuring before legs are set to the same height.

Drop down the legs, so that the seat is horizontal and position the fixing plate against the wall. The position of the fixing holes can now be determined.

## Shower Stool Safety

The shower stool should only be used as an aid for sitting. Do not stand on or mis-use the shower stool.

Do not exceed the weight limit of the seat, as stated on the product label.

Some users and children may require supervision whilst using the shower stool.

Always sit in the centre of the seat. Do not sit on the outer edges of the seat.

Fold the seat away, when not being used.

## Operation

To use the shower stool, fold down using the handle, located below the seat. DO NOT fold down using the seat, as this can cause damage to the device. Fold up using the handle, for storage.

Ensure that the seat is fully folded down and that the tips are secure on the floor/bath, before sitting on the seat.

## Care & Maintenance

The shower stool requires minimal maintenance. Periodically check the wall fasteners and pivot points are secure.

To clean the seat, occasionally wipe the seat with a soft cloth using a mild household detergent or disinfectant and warm water. Do not use harsh abrasives or bleach as these can cause damage and/or discolouration.