

# User Guide

## Easi-Riser Table – 3014/LO 3015/LO



### Introduction

Thank you for purchasing a Sidhil Easi-Riser over bed table.

Key Features include:-

Rising on underside load – Prevents the risk of trapped legs by automatically moving upwards.

Tilting top – Twelve angled settings plus flat position.

Split width top – Drinks may be placed on the level table section while the tilt table is raised.

Safety lever – Prevents accidental operation of the tilting top.

BioCote – Anti Bacterial coating

Wheeled base – Easy movement for helpers and users.

### Specifications



<b>3014/LO Easi-Riser Overbed table Standard Base</b>		<b>3015/LO Easi-Riser Overbed table – Wheelchair Base</b>	
Table top maximum Load	30Kg (66lb)	Table top maximum Load	30Kg (66lb)
Minimum table height	740mm (29.1")	Minimum table height	740mm (29.1")
Maximum table height	1090mm (42.9")	Maximum table height	1090mm (42.9")
Overall table area	910 x 380mm (35.8" x 14.9")	Overall table area	910 x 380mm (35.8" x 14.9")
Base width (internal)	Not Applicable	Base width (internal)	660mm (25.9")
Angle of tilt	0° to +47°	Angle of tilt	0° to +47°
Net weight	25.5Kg (56.2lb)	Net weight	25.5Kg (56.2lb)

### Assembly & preparing for use

Your table is delivered fully assembled and ready to use.

### Using your table: Do's & Don'ts

**DO** ensure you read these instructions and fully understand them before using your table.

**DO** ensure the table is cleaned on a regular basis

**DO** ensure the table is inspected on a regular basis (minimum once a year recommended).

**DO NOT** exceed the safe working load

The table is designed to carry a maximum load of 30Kg that is evenly distributed across the table top. It is advantageous to place heavier items towards the supporting column end.

## Adjusting the table

### Raising

Gently push up on the table underside **and** lever, table top will rise up. It will automatically lock in the desired position when the lever is released. Ensure that excessive force is not applied to the table top or damage could be caused to fixings.

### Lowering

Support the weight of the table, squeeze upwards the front lever and lower the table to the desired position, release the lever.

### Raising the tilt



**WARNING** – Risk of burns.

Ensure there are no hot liquids or drinks on the table while the angle of tilt is adjusted.

Squeeze and hold in the red handled safety lever. Raise the tiltable table part by lifting the back edge. Once the desired angle is reached, lower the table top until the latch mechanism engages firmly.

Note: the safety lever can be released once the table is lifted from the initial horizontal position.

### Lowering the tilt



**WARNING** – Risk of trapped fingers.

Do not hold the right hand side of the tiltable table whilst lowering.

Raise the tiltable section past the top most position, and then lower the tiltable section down towards the horizontal position. Squeeze to red handled safety lever to allow the table top to enter a fully horizontal position.

## Care and maintenance

### Biocote



The painted parts of your table come with an antibacterial additive incorporated into the powder coated metalwork. Biocote™ inhibits the growth of bacteria on the surface of the table frame. It is effective against a wide range of both gram positive and negative bacteria as well as fungi. Examples of bacteria that Biocote™ is resistant to include:

Bacillus Subtilis	Staphylococcus Aureus (MRSA)
Escherichia Coli 0157	Streptococcus Faecalis
Salmonella Enteritides	Listeria Monocytogenes



Biocote™ is an aid to keeping your table infection free but it does not substitute the need for the table to be cleaned at regular intervals.

### Cleaning

All surfaces to be wiped down with a disposable cleaning cloth, moistened with a mild detergent and diluted in hot water (60°C). Extra care should be taken around areas where excess dirt or dust may gather. Rinse down with clean water and dry off with a paper towel.

Always ensure that the table is dry before re-using.

It is not recommended to use any bleach or surface cleaner that may scour the painted surface.

### Warranty

Sidhil Limited standard terms and conditions apply to all purchases; a copy is available from the address below. Your statutory rights are not affected.



**Sidhil Limited, Sidhil Business Park, Holmfield, Halifax, West Yorkshire HX2 9TN**

**Tel: +44 (0)1422 233030 Fax: +44 (0)1422 233040 Email: [info@sidhil.com](mailto:info@sidhil.com)  
Sales Tel: +44 (0)1422 233030 Sales Fax: +44 (0)1422 233040**

*A Member of the Siddall & Hilton Ltd Group of Companies*



INSTRUC-1314 - 1315 rev1  
7/11/13