



TO USE THE STOOL

Place on the floor then slide the front and rear legs apart. Flip down the chrome 'P' shaped bar (arrowed on photo below) to lock the legs apart. Fold down the seat and the stool should look like the photograph (left).



TO ADJUST THE SEAT HEIGHT

The perching stool has a number of predrilled holes to allow seat height adjustment without tools. To adjust the seat height, turn one of the tristar knobs anticlockwise until released. Remove the knob and repeat for the other side. Adjust the seat to the desired height, then reinsert one of the knobs and turn clockwise to secure. Repeat for the other side.



PERCHING STOOL (PS001)

The perching stool is suitable as a seat to provide support when doing household tasks such as ironing, DIY, etc. The weight limit on the perching stool is 120kg (19 stone).

It is constructed from strong tubular steel and has a comfort padded seat and back rest.

The stool has vinyl upholstery which can be wiped clean with a damp cloth.

TO FOLD UP THE STOOL

Flip up the chrome 'P' shaped bar (arrowed)

Lift up by the top of the stool and back leg should naturally fold in. Flip up the seat for storage.

WARRANTY

The device is covered by a one year warranty from date of purchase which covers design and manufacturing defects. It does not include accidental damage, wear and tear or use error.



Drive DeVilbiss Healthcare
Birkenshaw, BD11 2HW, GB
IFU Issued 02/18

