



## Anti-Tip Wheel fitting guide



Anti Tip wheels can be retro fitted to a number of the U-Go range of wheelchairs (namely the 310B/330B range, 420/422/424/426 range and the 007/008/009 range, with both Transit and Self-Propelled models)

All these models have pre-drilled holes to accommodate anti tip wheels, these holes can only be seen when the rear stepper tube is removed, which the anti-tip wheels locate into.



So, the first thing to do is remove the rear stepper tube rubber located at the rear of the wheelchair above. The easiest way to remove the rear stepper tube is with the aid of a Stanley knife or equivalent (please take care). Make an incision in the plastic to reduce its rigidity which then allows the stepper tube to be slid off and removed.

Once the stepper tube is removed the pre-drilled holes can be seen. Undo the nut and bolt on the anti-tipper and then insert the anti-tipper onto to the metal frame where the stepper tube was located and replace the nut and bolt and tighten.



The anti-tip rear wheels can be height adjusted by pulling the silver circular button.