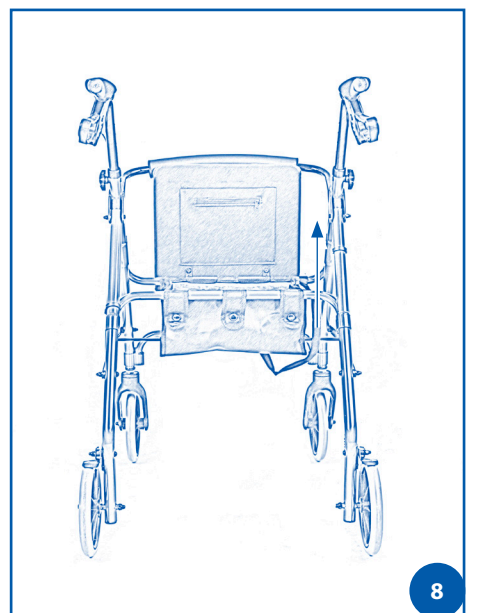
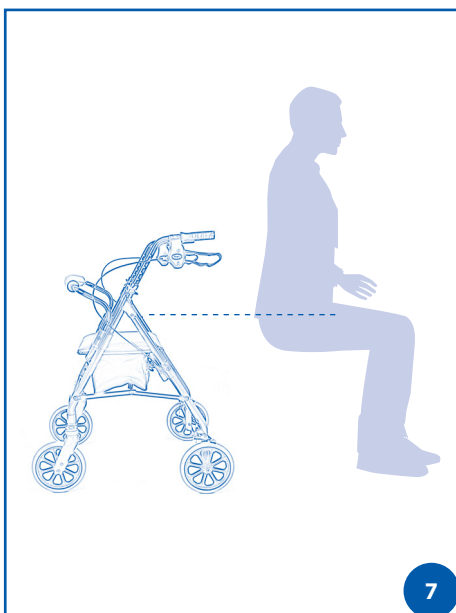
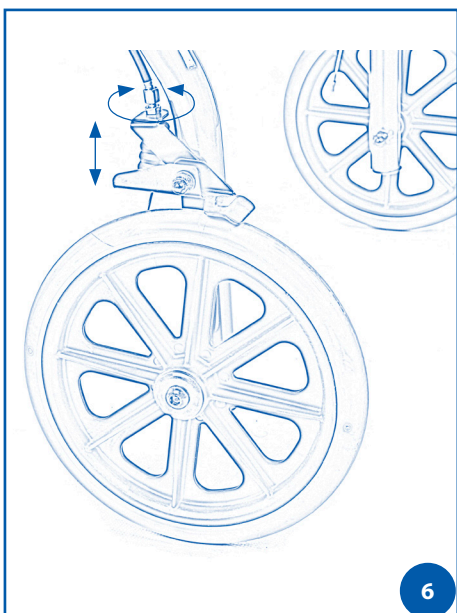
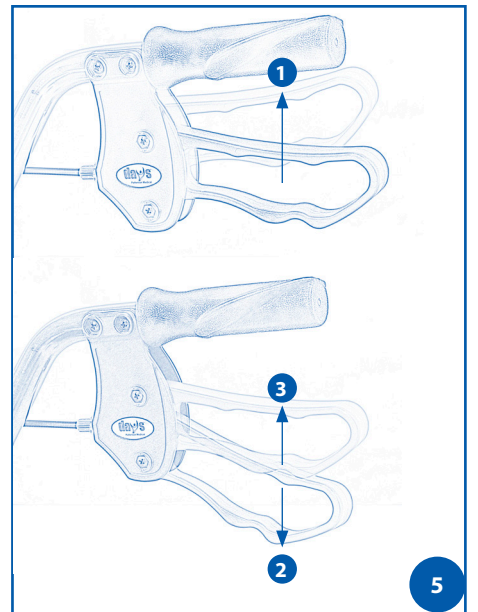
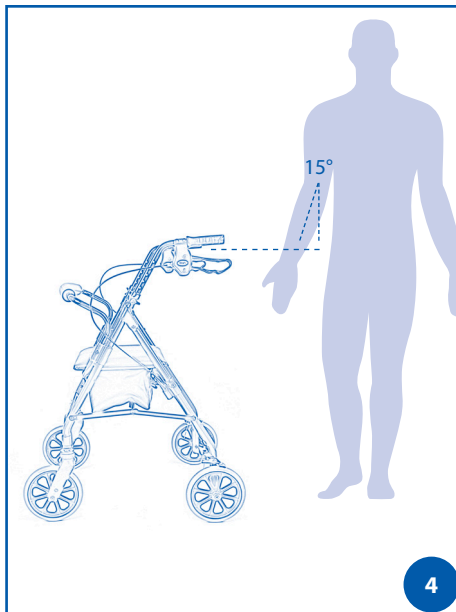
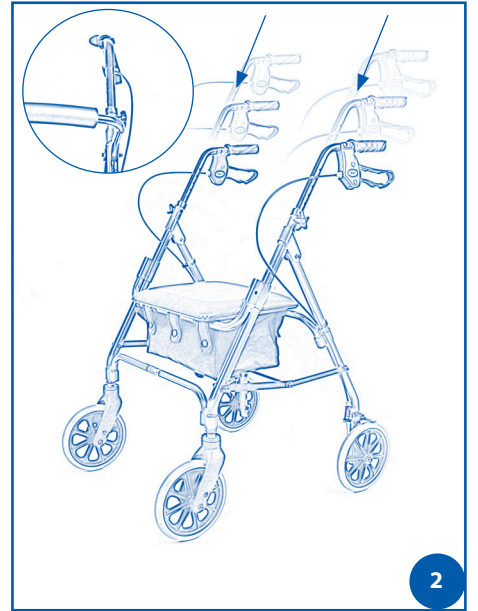
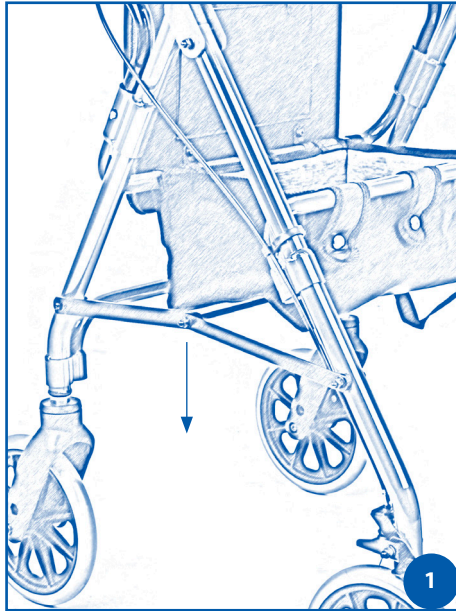
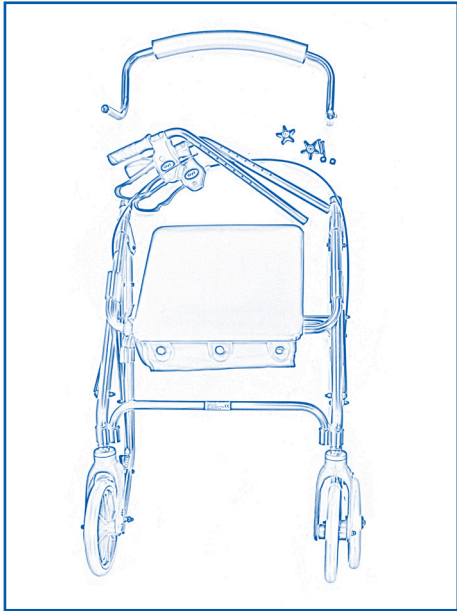
Days 100 Series Lightweight Rollators

Owner's Manual

EN Days 100 Series Lightweight Rollators
 FR Déambulateurs à roulettes légers Days série 100
 DE Leichte Rollatoren der Days 100-Serie
 ES Andadores Days ligeros – Serie 100
 IT Deambulatori Leggeri Days Serie 100
 NL Days 100 serie lichtgewicht rollators
 PT Andarilhos leves da série Days 100
 EL Days Σειρά 100 ελαφροί τροχήλατοι περιπατητήρες
 DA Days 100-serie Letvægtsrollatorer
 FI Days 100 -sarjan kevyet rollaattorit
 SV Days 100-serie med lättviktiga rullatorer
 NO Days 100-serien lette rullatorer
 SL Lahki rolatorji serije Days 100
 LT „Days 100“ serijos lengvos vaikštynės su ratukais
 PL Lekkie podpórki inwalidzkie Days z serii 100
 LV Days 100 sērijas vieglie rolatori
 ET Days 100 seeria kergekaalulised rulaatorid

	102	103	105	106
REF	091356302	091553130	091356328	081561661
	091553049	091553114	091356336	091553197
	091553072	091553098	091546563	091553213
	091553023	091553106	091553163	091553189
	091566579	091553148	091553171	091553221
	091553080		091553155	091566595
			7023389	091553205

EN	FR	DE	ES	IT	NL
Available Colours:	Disponibles dans les coloris suivants:	Erhältliche Farben:	Colores disponibles:	Colori disponibili:	Beschikbare kleuren:
Ruby Red	Rouge rubis	Rubinrot	Rojo rubí	Rosso rubino	Robijnrood
Blue	Bleu	Blau	Azul	Blu	Blauw
Racing Green	Vert gazon	Racing Green	Verde inglés	Verde corsa	Donkergroen
Quartz	Quartz	Quarz	Cuarzo	Quarzo	Kwarts
Purple	Violet	Violett	Púrpura	Viola	Paars
Russet Orange	Rouge grenadier	Rostorange	Naranja rojizo	Arancio ruggine	Donkeroranje
PT	EL	DA	FI	SV	NO
Cores disponíveis:	Διαθέσιμα χρώματα:	Tilgængelige farver:	Saatavilla olevat värit:	Tillgängliga färger:	Tilgjengelige farger:
Vermelho rubi	Ruby Red	Rubinrød	Rubiinipunainen	Rubinröd	Rubinrød
Azul	Μπλε	Blå	Sininen	Blå	Blå
Verde	Racing Green	British Racing Green	Tummanvihreä	Racinggrön	Racing-grønn
Quartzo	Quartz	Quartz	Kvartsi	Mörkblå	Kvarts
Roxo	Μωβ	Lilla	Violetti	Lila	Lilla
Laranja	Russet Orange	Russet Orange	Rusehtava oranssi	Rödbrun	Russet-oransje
SL	LT	PL	LV	ET	
Razpoložljive barve:	Galimos spalvos:	Dostępne kolory:	Pieejamās krāsas:	Saadaolevad värvid:	
Rubinasto rdeča	Rubino raudonumo	Rubinowa czerwien	Rubinsarkans	Rubiinpunane	
Modra	Mėlyna	Niebieski	Zils	Sinine	
Zelena »racing green«	Tamsiai žalia	Zieleń angielska	Tumši zaļš	Võidusõiduroheline	
Kremenasta	Kvarco	Kwarc	Kvarcs	Kvarts	
Vijolična	Violetinė	Fiolet	Violets	Lilla	
Oranžna »russet«	Rausvai oranžinė	Rudopomarańczowy	Oranžs	Ruske oranž	

Days 100 Series Lightweight Rollators

Introduction

The Days Lightweight Rollator is a durable walker manufactured from painted aluminium tubing with four solid tyre wheels. It incorporates a rest seat, storage bag and locking brakes. This walker also has height adjustable handles and can be folded for storage or transportation.

Before using your new Days Lightweight Rollator, it is essential that you read and fully familiarise yourself with the instructions for use.

Specifications

	102 X-Small	103 Small	105 Medium	106 Large
Overall Height	755-875mm	790-910mm	825-945mm	875-995mm
Overall Width	620mm	605mm	620mm	620mm
Overall Depth	640mm	655mm	655mm	670mm
Seat Height	505mm	540mm	545mm	630mm
Seat Width	350mm	350mm	350mm	350mm
Seat Depth	320mm	320mm	320mm	320mm
Width between Arms	460mm	460mm	460mm	460mm
Backrest Height	750mm	800mm	800mm	870mm
Diameter of Wheels	150mm	200mm	200mm	200mm
Weight	6kg	7kg	7.2kg	7.3kg
Max. User Weight				

Assembly Instructions

1. Remove all packaging and examine all parts for signs of shipping damage. **Do not use Rollator if it is damaged or any parts are missing.**
2. Open the Rollator frame fully. Push the folding hinge until straight to secure the Rollator in the correct position. Make sure all wheels make contact with the floor and the Rollator is stable. (See Fig 1, page 2)
3. Insert a handle with the pre-assembled locking loop brake into each handle tube. Ensure that the brake cable runs down the outside of the frame. (See Fig 2, page 2)
4. Depress the pin clips on the backrest and slide it into the tubes at the front of the Rollator. You should hear a "click" as the pin clips lock into the securing holes. Ensure both sides of the backrest are in place with the pin clips protruding through the holes. (See Fig 3, page 2)
5. Remove the protective plastic covering from the tyres.

Setting the Handle Height

1. The handles have five settings at 25mm intervals.
2. To determine the correct height of the handle (See Fig 4, page 2);
 - Stand upright behind the Rollator with your shoulders relaxed and arms slightly bent at 15°.
 - Set the handle height to be level with your wrists
3. To adjust each handle to the desired height, line up the holes appropriately. Slide the bolts through the holes from the inside of the frame and screw the tightening knobs until secure.

4. Ensure that the bolt heads are countersunk within the holes, the tightening knobs are fully tightened and the handles are secure and stable.

Using the Walker

When using the Rollator be aware that stability is dependent on all four wheels being in contact with a flat surface. Care should be taken when negotiating uneven ground, inclines and kerbs for example, as stability will inevitably be reduced.

Hand Brake Operation (See Fig 5, page 2)

The Rollator is fitted with locking brakes for added security. Before use test the brake by rotating the rear wheels and applying the brakes.

Note: Always engage and disengage brakes on both sides

1. To apply the brakes, squeeze both brake handles.
2. To lock the brakes, push both brake handles down until the click securely into the locked position.
3. To release the brakes simply pull up on the brake handles until they click.

Adjusting the Brake Pressure (See Fig. 6, page 2)

If the brakes are too hard to apply, the pressure can be adjusted by reducing the tension in the spring above the brake shoe.

- The pressure needed to brake can be adjusted by turning the knurled knob whilst holding the hexagonal nut below it.
- Both the knob and nut are located above the brake shoe.

- Turning the knob anti-clockwise increases the pressure needed to brake by moving the brake shoe nearer to the wheel and compressing the spring.
- Turning the knob clockwise decreases the pressure needed to brake.

CAUTION

Brakes that are adjusted to give light pressure will be less effective at preventing rear wheel movement under load.

Using the Rest Seat (See Fig. 7, page 2)

- Before using the walker as a rest seat both brakes must be applied and the frame pushed forward to check that the brakes are fully engaged. The brakes only lock the back wheels so the frame will not be completely immobile.
- When seated place both feet firmly on the ground and do not lean backwards or sideways.
- Do not use the seat without the safety backrest
- Only use the seat on level surfaces
- Do not exceed the maximum user weight

Folding for Storage or Transportation (See Fig. 8, page 2)

The rollator is easily folded by lifting the seat and pulling the black strap attached to the hinge and the upper crossbar.

Pushing down on the handles unfolds the Rollator.

Safety Precautions

- Ensure the Rollator is fully open, the seat is down and the locking hinge is straight to secure the Rollator in the correct position
- Ensure both handles are at the same height and that the tightening knobs are fully tightened. Test the Rollator for stability before applying full weight to the handles.
- Test the brake mechanism before use. When the wheels are locked, the brake pad should prevent the wheels from turning. If the wheels turn while the brakes are locked, refer to Adjusting the Brake Pressure section. When the brakes are released, the wheels should turn freely.
- The Rollator is a walking aid only and should not be used as a transportation device. Do not allow anyone to sit on the Rollator while it is being pushed
- Do not leave the Rollator unattended without locking the brakes
- Do not use the Rollator as a seat in any type of motor vehicle
- Always put the brakes in the lock position before sitting on the Rollator
- Do not stop, turn or reverse on a ramp or incline
- Do not use the seat when the Rollator is on an incline or uneven ground
- Do not make sharp turns

Care and Maintenance

1. The Rollator should be inspected before each use to ensure it is free from damage and excessive wear.

Check that:

- The brakes are operating effectively
- The hand grips are not loose or damaged
- The brake cables are not damaged in any way
- The wheels are securely in place and are not damaged or excessively worn
- The frame is not cracked or dented
- The frame is secure, stable and all nuts and bolts are tight

If you do find wear or damage to your Rollator, DO NOT USE.

2. Check that all wheels and folding parts move freely and that there is no build-up of debris around the axle
3. Keep the Rollator clean by wiping with a damp cloth on a regular basis. Do not use abrasive cleaning materials
4. If required, only use genuine parts supplied by Performance Health International Ltd or an approved stockist

Warranty

Performance Health International Ltd warrants that the Rollator is free from defects in material and workmanship. The warranty shall remain in effect for **3 years** from the date of original consumer purchase.

The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement such as brakes, wheels, handgrips and bags.

This warranty does not cover the following:

Replacement parts supplied by anyone other than Performance Health International Ltd or the stockist;

Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use;

Performance Health International Ltd is not liable for incidental or consequential injury or damage to property.

Should it be necessary to return the rollator, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise Performance Health International Ltd cannot accept any responsibility for transit damage to the unit.

This does not affect your statutory rights.

Please retain these instructions for future reference.

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These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Performance Health
International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield,
Nottinghamshire,
NG17 2HU UK
Tel: 03448 730 035
www.performancehealth.co.uk
International:
+44 1623 448 706

Distributed in the USA by:
Performance Health Supply, LLC
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012 USA
+1 800-323-5547
www.performancehealth.com

Distributed in Australia by:
Ausmedic Australia, Trading as
Performance Health ANZ
Unit 3, 3 Basalt Road
Pemulwuy NSW, 2145, Australia
1300 473 422
www.performhealth.com.au

Distribué en France par:
Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex, France
00(33) 03 10 00 79 30
www.performancehealth.fr

CEpartner4U
Esdoornlaan 13
3951 DB Maarn
The Netherlands
www.cepartner4u.com

Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex, France

Days® is a registered trademark of Performance Health International Ltd.