

M66739

Silver

P64612

Electric Blue

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The Compact Easy Rollator is designed to provide support and assistance to those unable to, or who lack the confidence to, walk without some level of support. This Rollator is intended for indoor and outdoor use.



2. SAFETY

Warning: The maximum user weight must not be more than 21 st (135 kg).

Do not adjust the rollator while in use.

Only use as a walking aid in the forward direction.

Do not use the rollator on stairs or escalators and take care on steep hills.

Do not sit on the seat without locking the brakes first.

Do not use the seat to transport people or objects.

Do not use the seat without the backrest in place.

Do not hang items from the handles as this will affect the stability of the rollator.

Only use the specified accessories and spare parts for this rollator.

Make sure the rollator is locked in the open position before use.

Loads should only be transported using the bag supplied (maximum load 5 kg).

Handle Height Range: 805 - 935 mm

Width: 220 mm (folded) - 650 mm (Unfolded).

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Maximum User Weight




Contents:

1. Compact Easy Rollator
2. Bag
3. Bag Handle
4. Backrest.

4. ASSEMBLY

4.1 Unfolding the Rollator:

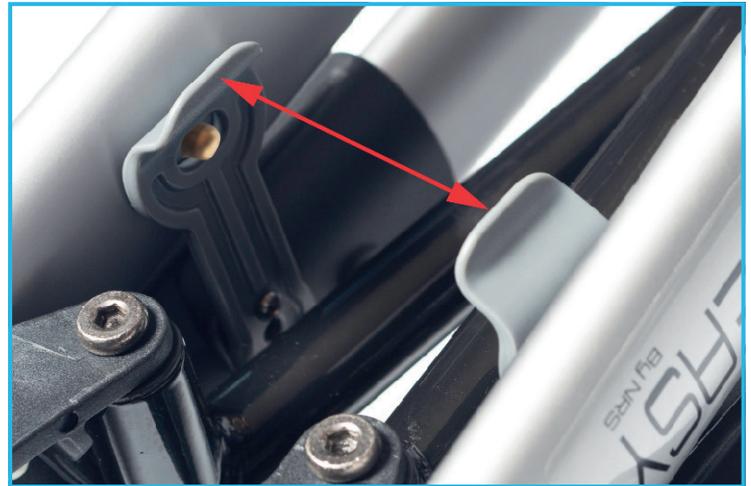


1. Squeeze the handle clips together to release the handles from the frame.


Note: Do the above step from below while the rollator is in the upright position (see below).



2. Lift the handles to an upright position making sure the handle bracket collar fits securely over the locking bracket.

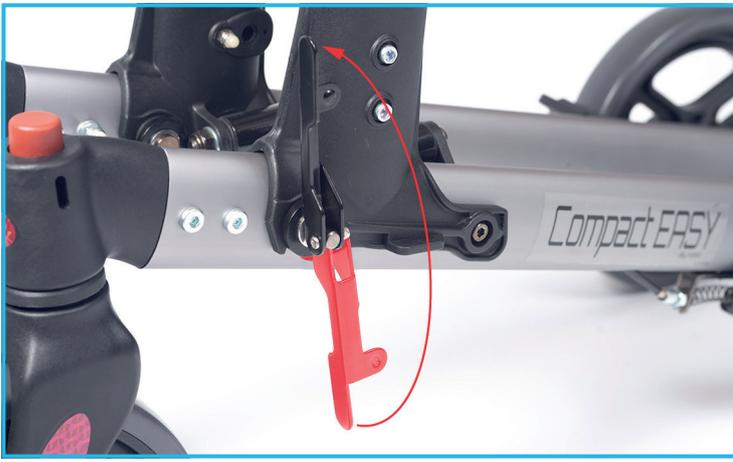

3. Pull the locking lever towards you, rotating through 180 degrees, to lock the locking bracket washer against the handle bracket collar.

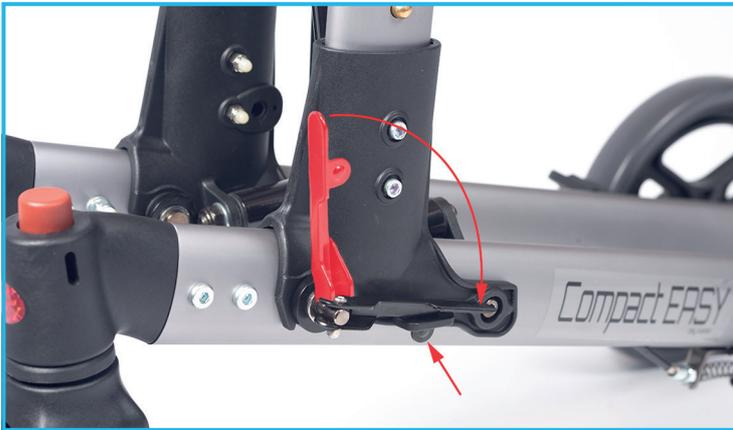

4. Rotate the locking lever 90 degrees, making sure the lever locks into place as shown.
5. Repeat for the opposite side.



6. Undo the seat frame clips.



7. Push down on the seat tubes until they click into place.

Note: Keep fingers from under the seat tube.

The Rollator is now fully open.

4.2 Fit the Back Rest

Fitting the backrest



1. Slide the back rest onto the handle bracket.



2. Use a Phillips screwdriver to fasten the seat rest bracket to the handle bracket.
3. Repeat for the opposite side.

Removing the backrest

Reverse the fitting process in Section 4.2.

4.3 Fitting the Bag

Fitting the Bag



Slide the bag fixing loops onto the seat tubes, making sure the loops fit under the seat tube hooks.

Note: Make sure the bag is fully secure before use.

Removing the Bag

Reverse the fitting process in Section 4.3.

4.4 Handle Height Adjustment

Adjust the Rollator handle height so that when the user's arm is down to their side, the hand-grip is at wrist height. This will ensure the arms are at an approximate 20°-30° bend when using the Rollator.



1. Push and hold the handle button-clip, then raise or lower the handle until the button-clip clicks into the required position.


Note: Make sure that the button-clip has fully engaged into the hole.

2. Repeat for the opposite handle.

Note: Make sure both handles have been set to the same height.

5. USING THE ROLLATOR

5.1 Brake Operation

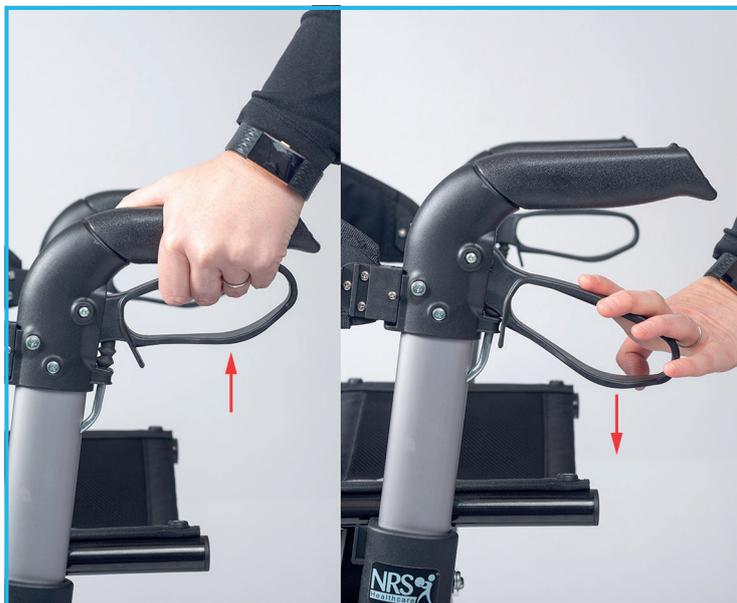

1. To slow down or stop the rollator, pull both brake handles up.

2. To apply the parking brake, push the brake handles down until they lock into place.

Note: Make sure both brake handles have been applied.

3. Pull both handles up to release the parking brakes.

Warning: If the brakes fail to stop or lock the wheels, stop using the rollator immediately. Do not use the rollator again until the brakes have been checked and correctly adjusted. Refer to Section 5.2.

5.2 Brake Adjustment

Warning: The following section requires some technical ability, do not attempt to carry out the adjustment unless you are confident to do so. Incorrectly adjusted brakes can lead to injury, please contact your supplier if you require assistance.



Tighten the Brakes

While holding part (A), use a 10 mm spanner to tighten the locking nut (B) in a clockwise direction.

Note: You may need to use a pair of pliers to hold part (A) still during the adjustment.

Loosen the Brakes

While holding part (A), use a 10 mm spanner to loosen the locking nut (B) in an anti-clockwise direction.

Note: You may need to use a pair of pliers to hold part (A) still during the adjustment.

Note: There must be a gap of 2-3 mm between the brake mechanism and the wheel, when the brake lever is released.

Test the brakes in accordance with Section 5.1.

5.3 Folding the Rollator

Note: Make sure all items have been removed from the bag before folding.



4. Lift the seat handle until the seat tubes release from the brackets.
5. Reverse the unfolding process (refer to Section 4.1)

5.4 Using the Walking Stick Holder



Place the walking stick in the cup, then clip into place.

6. MAINTENANCE

Check all button clips are fully clicked into position.

Check all wheels can rotate freely.

Regularly check for wear and damage. Any suspected faults or damage should be reported immediately to your supplier and the product taken out of use.

Check brakes regularly in accordance with Section 5.1 before use.

7. CLEANING

Make sure that all dirt and debris is removed from around the wheels, axle and brakes.

To clean the bag wipe with a damp cloth. Do not machine wash.

Clean the frame regularly, using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Always allow the product to dry thoroughly prior to use or storage.

8. ACCESSORIES & SPARE PARTS

P66736 Replacement Bag & Backrest.

For more details please visit our website:

www.nrshealthcare.co.uk or call Customer Services.

9. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

10. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

UK CA NRS HEALTHCARE Sherwood House,
Cartwright Way, Forest Business Park, Bardon
Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

CE NRS HEALTHCARE 4C Altona Road, Blaris
Industrial Estate, Lisburn, Co Antrim,
Northern Ireland BT27 5QB

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk