

Product Technical Data Sheet 057 Mobility Care Aluminium Rollator

Product Range

M39634 Aluminium Rollator

Product Description & Intended Purpose

This Rollator is designed to provide support and assistance to those unable to, or who lack the confidence to, walk without some level of support. This rollator is intended for indoor and outdoor use.

Product Features

- Lightweight aluminium frame.
- Large diameter, puncture proof wheels.
- Lockable, loop brake system, easy to operate.
- Height adjustable handles.
- Padded seat and back for added comfort.
- Basket for storage and to aid carrying bags etc.

Technical Specification

Dimensions (mm)

- Height: 770 –910
- Overall width: 640
- Folded width: 230
- Width between handles: 450

- Seat Height: 530
- Seat Size: 345 X 320
- Wheel Diameter: 200 (8")

Maximum User Weight: 20st (127kg)

Product Weight: 18.5lbs (8.4kg)

Installation Information: (Note: for detailed information refer to User Instructions)

- Before assembly consult a suitably qualified healthcare professional regarding the correct height adjustment for maximum support and correct brake activation.
- Unfold the rollator and push down on the seat until all four wheels are extended.
- Position the backrest bar such that the padded portion faces forwards. Secure in place by pushing into the brackets until the buttons click.
- Insert the handlebars into the main rollator frame. Set to the desired height and secure using the hand wheel.
- Fit the basket by hooking onto the frame below the seat.



Safety Precautions

- Do not exceed the maximum user weight , 20st (120kg).
- Only use the rollator as a walking aid in a forward direction.
- Do not use the rollator on stairs or escalators.
- Do not attempt to move the rollator when the seat is occupied.
- Ensure the rollator is locked in the open position before use.
- Ensure the brakes are locked before using the seat.
- Take care when using the rollator with a load in the basket as it may become unbalanced.

Cleaning

- Clean regularly, use a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Ensure all dirt and debris is removed from around the wheels, axle and brakes.

Care & Maintenance

- Check all screws and knobs are securely fastened.
- Check the brakes are operating correctly. See User Instructions for more details.
- Check for general wear or damage. Any suspected fault or damage should immediately be reported to the supplier and the product taken out of action.

FAQ's

• Is this suitable for indoor and outdoor use?

Yes the rollator can be used on both indoor and outdoor surfaces, however, we advise you ensure there is plenty of space before using it inside.

• How do I measure the correct height for the handles?

Stand with your feet flat on the floor, in your usual footwear, with your arms held loosely at your sides and measure the height from the floor to your wrist. This will be the optimum height for the rollator handles.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK